

3. Injection

Administer medication in accordance with established protocol. For user convenience, the needle "bevel-up" position is oriented to the lever arm.



4. Activation

Activate safety mechanism immediately after injection by pushing the lever arm forward until needle tip is completely covered.

NOTE: Activate away from self and others. Visually confirm needle tip is fully covered.

Discard after single use in an approved sharps collector.



Don't Forget...

Aseptic technique and proper skin preparation of the site are essential. Follow your institution's policy for safe disposal of all medical waste.

BD SafetyGlide™ Needle

1mL Syringe with Detachable BD SafetyGlide™ Needle		
Catalog #	Gauge/Needle	Packaging
305927	27G x 3/8"	50/Box – 400/Case
305903	25G x 3/8"	50/Box – 400/Case
3mL Syringe with Detachable BD SafetyGlide™ Needle		
305904	25G x 3/8"	50/Box – 400/Case
305924	25G x 1"	50/Box – 400/Case
305905	23G x 1"	50/Box – 400/Case
305923	22G x 1"	50/Box – 400/Case
305906	22G x 1 1/2"	50/Box – 400/Case
305909	21G x 1 1/2"	50/Box – 400/Case
5mL Syringe with Detachable BD SafetyGlide™ Needle		
305907	22G x 1 1/2"	50/Box – 400/Case
10mL Syringe with Detachable BD SafetyGlide™ Needle		
305908	22G x 1 1/2"	50/Box – 400/Case
BD SafetyGlide™ Needle		
305921	27G x 3/8"	50/Box – 500/Case
305901	25G x 3/8"	50/Box – 500/Case
305916	25G x 1"	50/Box – 500/Case
305902	23G x 1"	50/Box – 500/Case
305909	22G x 1"	50/Box – 500/Case
305900	22G x 1 1/2"	50/Box – 500/Case
305915	21G x 1"	50/Box – 500/Case
305917	21G x 1 1/2"	50/Box – 500/Case
305918	18G x 1 1/2"	50/Box – 500/Case
BD SafetyGlide™ Needle & Syringe		
Catalog #	Gauge/Needle	Packaging
Insulin Syringe with Permanently Attached Needle		
305930	1mL 29G x 1/2"	100/Box – 400/Case
305932	1/2 mL 29G x 1/2"	100/Box – 400/Case
305934	1/2 mL 30G x 3/16" Thinwall	100/Box – 400/Case
305935	3/10 mL 29G x 1/2"	100/Box – 400/Case
305937	3/10 mL 31G x 3/16" Thinwall	100/Box – 400/Case
1mL Tuberculin Syringe with Permanently Attached Needle		
305945	27G x 1/2"	100/Box – 400/Case
305946	26G x 3/8" (IB)	100/Box – 400/Case
1mL Allergist Tray with Permanently Attached Needle		
305950	27G x 1/2"	25/Tray – 1000/Case
305951	26G x 3/8" (IB)	25/Tray – 1000/Case

For more information call
1.888.237.2762

BD, BD Logo, and all other trademarks are property of Becton, Dickinson and Company. © 2013 BD.

BD 3/13 - MSS0084



BD Medical
1 Becton Drive
Franklin Lakes, NJ 07417

www.bd.com



Helping all people
live healthy lives

BD SafetyGlide™ Shielding Hypodermic Needle

How to Use the BD SafetyGlide™ Needle

1. Attachment

Attach the BD SafetyGlide™ needle to any standard BD Luer-Lok™ or luer slip hypodermic syringe. Twist until firmly seated. Pull shield straight off needle to avoid damaging needle point.



2. Aspiration

Draw up medication in accordance with established protocol.



Helps to Meet a Wide Variety of Clinical Needs




The BD SafetyGlide™ Shielding Hypodermic Needle is a single-handed, needle-based safety device that can be used with any size conventional syringe.

It is complemented by the BD SafetyGlide™ Syringe for Insulin, TB and Allergy which is recommended for small doses of medication.



The BD SafetyGlide™ Shielding Hypodermic Needle Is Designed for Efficiency, Safety and Patient Comfort

Prior to Activation

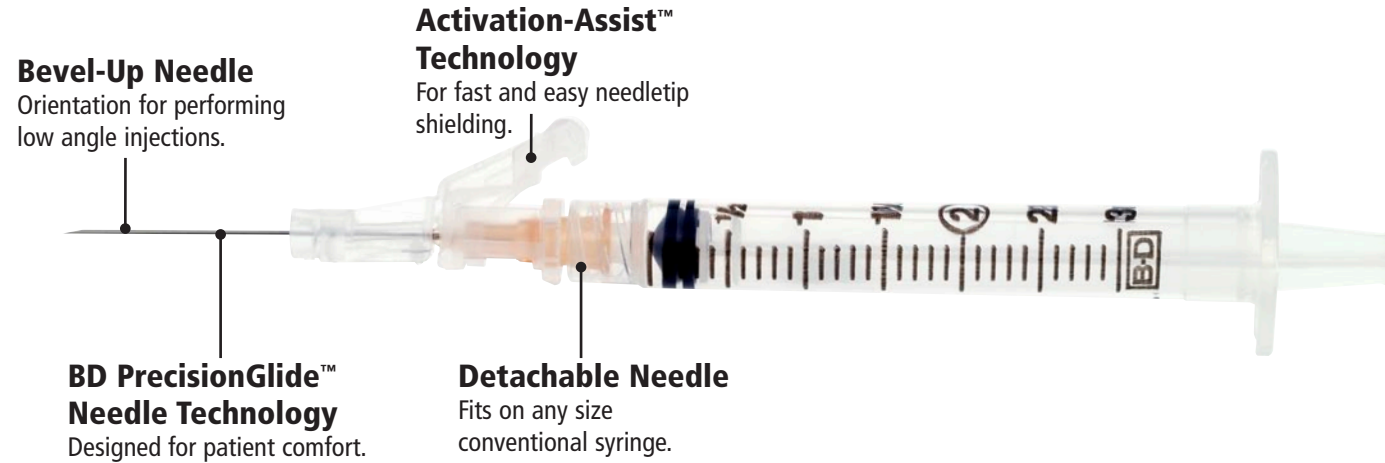

After Activation



Features and Benefits for Healthcare Workers and Their Patients

BD PrecisionGlide™ Needle Technology

- Designed for patient comfort
- Leading brand of hypodermic needle technology in the U.S.

One-Handed Activation

- Virtually no change in technique
- Requires minimal training

Proven Dependability

- Secure and visible lock
- Virtually no splatter upon activation

Detachable Needle

- Allows clinicians to change needles for aspiration and administration

Designed for Efficiency

- Detachable needle fits on any size conventional syringe, which helps to facilitate standardization and SKU reduction