

Shaping the Future of Safety

BD Integra™ Syringe with Retracting BD PrecisionGlide™ Needle



IMPORTANT USAGE GUIDELINES

1. ASPIRATION



Draw up medication in accordance with established protocol.

2. INJECTION



Administer medication in accordance with established protocols, making sure all medication has been dispensed.



3. ACTIVATION*



Push plunger rod to activate the safety mechanism. Depress until 2 "clicks" are sensed.

***Important:**

Activation of device can occur before or after needle is withdrawn from patient.

4. NEEDLE RETRACTION



The needle will disappear into the syringe and the plunger will be recessed into the barrel when activation is complete. There is no exposure to the used needle.

Discard immediately into nearest sharps container.

NEW ISO NEEDLE HUB¹/ PACKAGING CHART

25G

23G

22G

21G

BD INTEGRA BLUNT FILL NEEDLE²

18G

THINGS TO REMEMBER:



- BD Integra Needle and Syringe **DO NOT** HAVE A STANDARD LUER CONNECTION.
- When pressurizing a medication vial, do not continue to press the plunger rod past the point of resistance, as this may activate the device prematurely.
- If Activation is performed before needle is withdrawn from patient, do not advance needle while depressing plunger rod.
- If Activation is performed after needle is withdrawn from patient, minimal splatter may occur. Activate away from self and others.
- **DO NOT** use this product to draw blood or for venipuncture.

¹ New colors reflect worldwide ISO color standards to enhance worldwide sourcing and product supply.

² Not for injection