




	NE-C28 COMPRESSOR NEBULIZER	NE-C30 COMPACT COMPRESSOR NEBULIZER	NE-U22V PORTABLE NEBULIZER with V.M.T.®
			
Medication Capacity	7 mL	7 mL	7 mL
Nebulization Rate	Approximately 0.4 mL/min	Approximately 0.35 mL/min	Approximately 0.4 mL/min
Operation Liter Flow	3.5 lpm	3.8 lpm	N/A
Particle Size	MMD Approximately 5μ	MMD Approximately 5μ	MMAD Approximately 5μ
Batteries	N/A	Optional Accessory	2“AA” (Not Included)
AC Adapter	N/A (Plug in-wall unit)	Included	Included (Limited Time)
Limited Warranty	5 Years	5 Years	2 Years
Reimbursement (HCPC)	E0570	E0570	E0574
UPC Code	0 73796 45328 2	0 73796 45330 5	0 73796 45122 6

Q. Are all of these units portable?

A. The NE-C28 is a compressor nebulizer that plugs into an AC outlet. The other units have AC adapters and battery options. All units are packaged with carrying bags or cases and the accessories needed to take a treatment.

Q. How often should I clean the MicroAir® NE-U22V?

A. Wash the medication bottle, mesh cap, mask, mouthpiece, and mask and mouthpiece adapter with distilled water after every use. Disinfect after the last treatment of the day. (See Instruction Manual for details)

Q. Are the batteries for the NE-C21V and the NE-C30 interchangeable?

A. No, each unit has its own specially designed rechargeable battery for optimal performance.

Q. Where can patients purchase Omron nebulizers?

A. U.S. Federal Law restricts these devices to sale on the order of a physician or licensed healthcare provider. Prescriptions can be filled by qualified providers.

	PF9940 PEAK FLOW METER	C900 REUSABLE NEB KIT	C920 and C922 ADULT and CHILD MASK	C30BATSET RECHARGEABLE BATTERY AND BATTERY CHARGER
				
Nebulizer To Be Used With	N/A	NE-C28 and NE-C30	NE-C28, NE-C30 and NE-U22V	NE-C30
Description	<ul style="list-style-type: none"> • Helps monitor asthma • Accurately measures peak flow 	<ul style="list-style-type: none"> • Includes Neb kit, Air Tube and Mouthpiece • Replace every 6 months 	<ul style="list-style-type: none"> • Universal design for use with different nebulizers • Adjustable straps allow for a secure yet comfortable fit 	<ul style="list-style-type: none"> • Includes rechargeable battery and battery charger • Charge before using with the NE-C30 unit
Features	<ul style="list-style-type: none"> • Dual flow range for adults and children • Compact design can be used anywhere • Can be used with disposable mouthpiece 	<ul style="list-style-type: none"> • Uses VVT (Virtual Valve Technology) for optimal performance without valves • Neb kit is dishwasher safe • Sturdy polypropylene construction 	<ul style="list-style-type: none"> • Metal band over nose reduces leakage • Clear plastic makes it easy to monitor patient • Adult and child sizes meet patient needs 	<ul style="list-style-type: none"> • Battery life approximately 300 charging cycles • Charge lasts approximately 35 minutes • Battery clicks to bottom of unit for ease of use
UPC Code	0 73796 69940 6	0 73796 39000 6	C920 - 0 73796 39200 0 C922 - 0 73796 39220 8	0 73796 30290 0

Q. How often should the C900 NEB KIT be replaced when using the NE-C28 and NE-C30?

A. The reusable Neb kit should be replaced every six (6) months.

Q. Will the Peak Flow Meter work for patients of all ages?

A. The PeakAir® Peak Flow Meter measures from 50-750 liters per minute, which meets the need of most patients.

Q. Will the masks (C920 and C922) fit the MicroAir® (NE-U22V)?

A. Yes, the masks are a standard size and will fit almost any nebulizer, including the MicroAir®. The NE-C21V has its own unique mask.

Q. Is there a nasal delivery option available?

A. Omron's nosepiece, #C911, is available separately.

Omron Healthcare Consumer Support can assist your patients with questions regarding the purchase or use of the unit. The toll free number for consumers is (800) 634-4350 and for Professional Medical (877) 216-1336. Automated information is available 24 hours a day, 7 seven days a week.

Customer Service Representatives are available 8:30 am to 4:30 pm CST, Monday through Friday.